








PRODUCT CATALOG
MEDICAL COMPRESSION
STOCKINGS

ALL PRODUCTS AT A VIEW

						
VenoTrain look	HONEY					
	CARAMEL					
	BLACK	●	●	●	●	
	SILVER					
micro	CREAM					
	HONEY					
	CARAMEL					
	AMARETTO					
	ALMOND	●	●	●	●	
	ESPRESSO					
	BLACK					
	ANTHRACITE					
	NAVY					
	discrétion	CLAIR				
BEIGE						
ESPRESSO		●	●	●		
BLACK						
JAVA						
business	WHITE					
	TERRA					
	BLACK	●				
	GRAPHITE					
	NAVY					
soft	NATURAL					
	CARAMEL					
	ESPRESSO	●	●	●	●	
	BLACK					(custom-made)
	NAVY					
soft S	NATURAL					
	CARAMEL					
	ESPRESSO					
	BLACK					
	NAVY					
clinic	NATURAL	●	●	●		
impuls	CARAMEL	●	●	●		
	BLACK					(custom-made)
ulcertec	NATURAL	●				
soft Arm sleeves	NATURAL					
	CARAMEL					●
	BLACK					
delight	CARAMEL					
	BLACK					
curaflow	CREAM					
	CARAMEL					
	BLACK					
	NAVY					
	BORDEAUX					
	ANTHRACITE					
act	SAND					
	CARAMEL					
	AMARETTO					
	ESPRESSO	●	●	●	●	
	BLACK					
	GRAPHITE					
	NAVY					
	NATURE					

VENOTRAIN MEDICAL COMPRESSION STOCKINGS: ALWAYS THE RIGHT CHOICE

Whether as a preventive measure, for standard phlebological cases, for advanced vein problems or even for the treatment of venous leg ulcers and lymphedema:

VenoTrain medical compression stockings cover all uses/indications associated with modern compression therapy and are also offer clinically proven effectiveness, unique material properties and a patient-oriented modern design.

The indication-driven treatments cover a wide range of round-knit and flat-knit styles in both custom-made and standard versions. The Perfect-Fit sizing system developed by Bauerfeind on the basis of digital leg measurements offers unique flexibility in ensuring that medical compression stockings from the standard product range fit perfectly.

All VenoTrain products satisfy the requirements of the German Medical Devices Act (Medizinproduktegesetz – MPG) and bear the RAL quality mark. The Oeko-Tex Standard 100 is a quality mark that guarantees particularly high levels of skin tolerance.



Colors may differ due to printing.
Please refer to the current price list for further styles.

CONTENT

Icons	3	VenoTrain ulcertec	22
Measurement technology: Guarantees optimal care for your customers	4	VenoTrain soft Arm	24
Systematic Approach to Compression	5	VenoTrain delight	26
VenoTrain look	6	VenoTrain curaflow	28
VenoTrain micro	8	Product Range (standard/custom-made)	30
VenoTrain discrétion	10	Sizing systems	32
VenoTrain business	12	VenoTrain act	34
VenoTrain soft	14	VenoTrain Accessories	36
VenoTrain soft S	16	Individual customer service	38
VenoTrain clinic	18	Order pads	39
VenoTrain impuls	20	Products and health care concepts: For better quality of life	39

FEATURES

	PERFECT-FIT SYSTEM		SHIMMER		GRIP-RELIEF-STRUCTURE
	MICROFIBER. CLIMA REGULATING OPTIMAL WEARING COMFORT		INNOVATIVE MIX OF MATERIALS		DEEP ACTION FOR THE VENOUS SYSTEM
	BREATHABLE		VERSATILE USE		EXTRA SOFT AREA BEHIND THE KNEE
	EFFECTIVENESS CLINICALLY PROVEN		EASY DONNING		POST-SURGICAL CARE
	2-STOCKING SYSTEM		90° HEEL		
	LINER WASHABLE AT 95 °C		EXTRA SOFT ELBOW AREA		

TOE

	Closed	
	Open	
		Diagonal cut

TOP BANDS

	Silicone dots		Silicone dot pattern		Honeycomb
	Lace		Lace sensitive		

MEASUREMENT TECHNOLOGY: GUARANTEES OPTIMAL CARE FOR YOUR CUSTOMERS



BODYTRONIC® 200

Leg health can now be measured quickly and easily thanks to the BODYTRONIC 200, the compact and portable system for measuring vein function.


With wireless transfer to a PC via Bluetooth, the measurement data can be used to devise preventive measures or obtain additional information for proposed medical treatments.



BODYTRONIC® 600

Individual measurements for medical compression stockings need to be highly precise if the proposed treatment is to be effective (quality and test specifications RAL-GZ 387:2008).

The BODYTRONIC 600 measurement system gives you the security of reliable 3D circumference values and a guaranteed fit and can even link up directly with Bauerfeind's online ordering system.

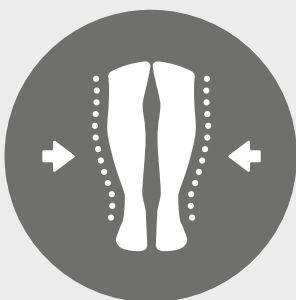

PERFECT-FIT SIZING SYSTEM


More than 500.000 legs have already been measured using Bauerfeind's measurement technology. These measurements form the basis for the Perfect-Fit system, which can help provide considerably more of your customers with compression stockings in standard sizes.

This reduces delivery times, simplifies logistics and lowers costs while ensuring items fit as well as possible – all of which benefits everybody.


SYSTEMATIC APPROACH TO COMPRESSION

Diagnostic findings	No visible signs of a venous disorder	Spider veins Reticular varices	Varicosis Imminent functional insufficiency	Varicosis with edema Functional insufficiency	Varicosis with trophic skin changes Atrophy blanche, dermatoliposclerosis	Healed ulcerus cruris venosum	Active ulcerus cruris venosum
CEAP-classification*	C0	C1	C2	C3	C4	C5	C6

Medical condition	Mild discomfort phlebological standard case				Severe discomfort phlebological severe case		
-------------------	---	--	--	--	---	--	--

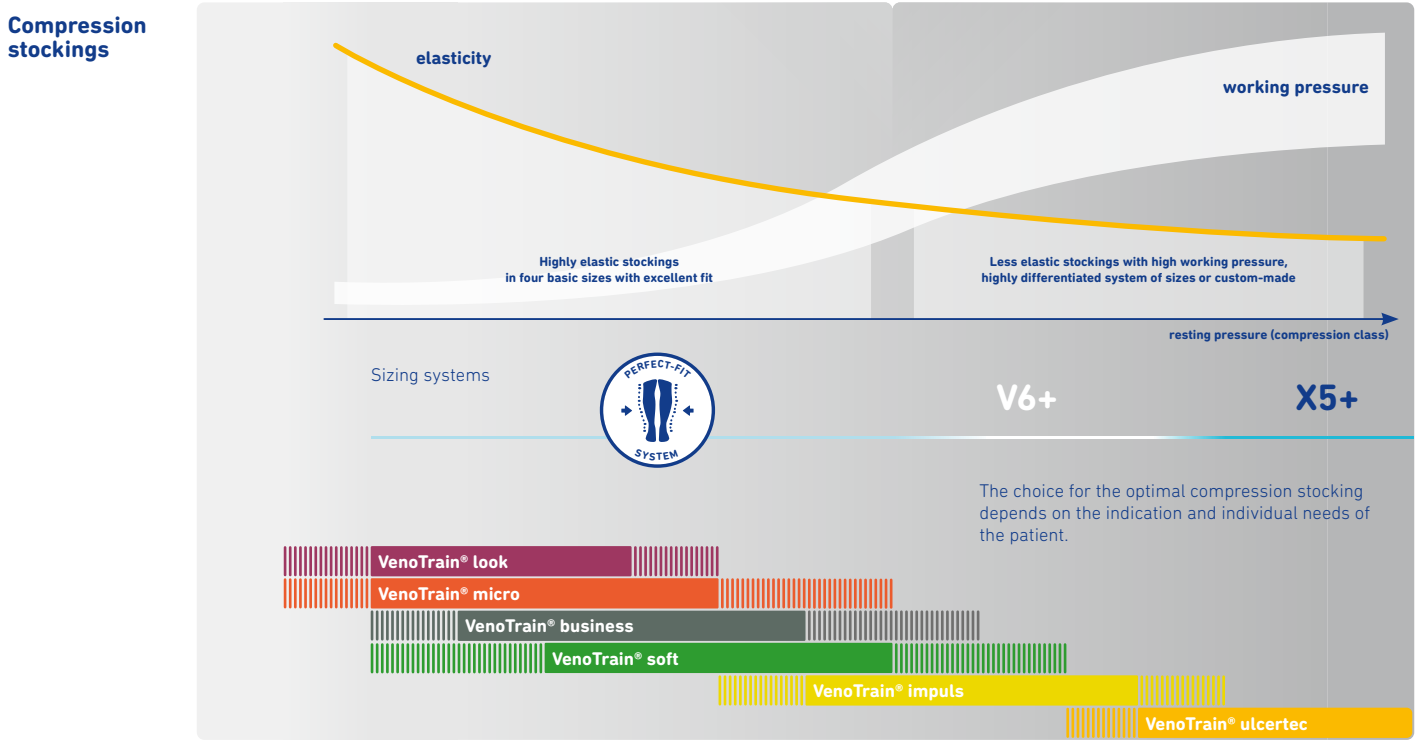

Compression classes	Ccl 1 18–21 mmHg	Ccl 2 23–32 mmHg	Ccl 3 34–46 mmHg
---------------------	----------------------------	----------------------------	----------------------------

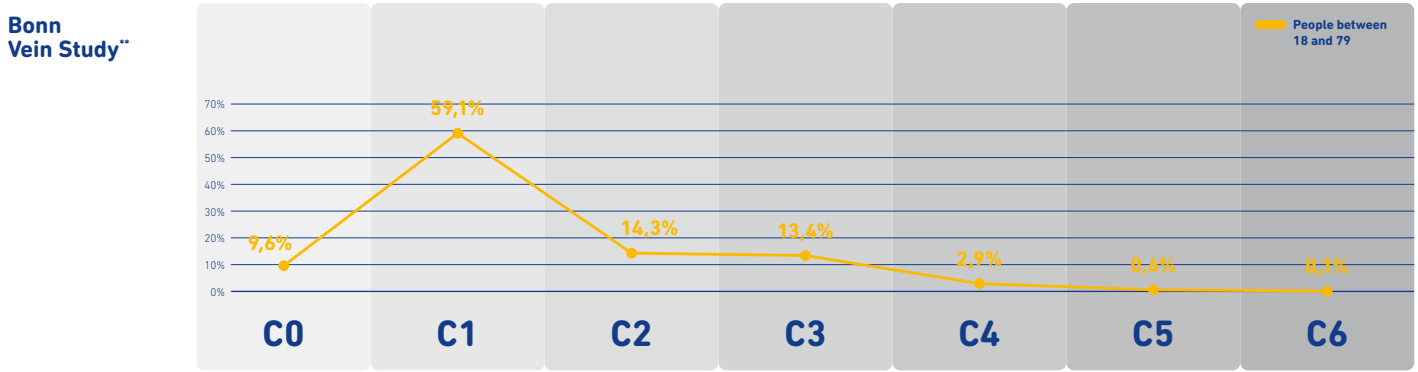

* Partsch, H.: Klassifikation von chronischen Venenerkrankungen heute, Vasomed 1/2003
 ** E. Rabe et al. Bonner Venenstudie der DGP, Phlebologie 1/2003

VenoTrain® look

QUALITIES

For everyone who loves something special!
The VenoTrain look is a real highlight among medical compression stockings, thanks to its glossy look: guaranteeing a stylish appearance on any occasion!

The VenoTrain look has a figure-enhancing push-up effect, while stretch comfort technology and special trilobal fiber guarantee a perfect fit.

- fashionable and elegant glossy effect
- slightly transparent

MATERIAL CONTENT

Ccl 1 60 % polyamide
40 % elastane

Ccl 2 55 % polyamide
45 % elastane

CARE INSTRUCTIONS



COLORS

HONEY

CARAMEL

BLACK

SILVER









STYLES





VenoTrain look OVERVIEW

SIZING SYSTEM	PERFECT-FIT (S–XL)
COMPRESSION CLASS	Ccl 1 (18–21 mmHg) Ccl 2 (23–32 mmHg)
TOE	closed

STANDARD SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AT	 ATU
Top band	●		●	●
Silicone dot pattern		●		

CUSTOM-MADE SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AT	 ATU
Top band	●		●	●
Lace		●		
Silicone dot pattern		●		

VenoTrain® micro

QUALITIES

With a particularly high microfiber content that makes it comfortable to wear, the VenoTrain micro is one of the most popular compression stockings.

Available in many designs and a wide selection of colors, it highlights the individual style of your customers.

Our individually colored batik versions in marine or black are particular eye-catchers.

- **50 % microfibers: breathable, kind to the skin and easy to clean**
- **figure-enhancing panty area (pantyhose) and comfort sole**
- **opaque full-fashioned stocking quality**
- **also available as a Design Edition with the patterns "Tango" and "Jive"**

COLORS AND DESIGN EDITION

	Tango	Jive	Batik Black	Batik Navy
CREAM				
HONEY				
CARAMEL				
AMARETTO				
ALMOND				
ESPRESSO				
BLACK				
ANTHRACITE				
NAVY				

MATERIAL CONTENT

Ccl 1 65 % polyamide
35 % elastane

Ccl 2 55 % polyamide
45 % elastane

CARE INSTRUCTIONS



VenoTrain-STYLIST

Select your favorite design quickly and easily:
www.bauerfeind.com/venotrain-stylist

As an added extra, all styles and colors of the VenoTrain micro can be optionally dyed over the top in navy or black using a batik technique. Delivered within 10 working days.

Colors may differ due to printing.









STYLES





VenoTrain micro OVERVIEW

SIZING SYSTEM	PERFECT-FIT (S–XL)
COMPRESSION CLASS	Ccl 1 (18–21 mmHg) Ccl 2 (23–32 mmHg)
TOE	open, closed

STANDARD SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AT	 ATU**
Top band	●		●	●
Lace sensitive*		●		
Silicone dots***	●	●		
Silicone dot pattern		●		

CUSTOM-MADE SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AT	 ATU**
Top band	●		●	●
Lace sensitive*		●		
Silicone dots***	●	●		
Silicone dot pattern		●		

* From May 1, 2017, the following will apply to VenoTrain micro: The Lace Sensitive silicone top band will replace the Lace silicone top band. The Lace Sensitive silicone top band will be available only for the Design Edition and fashion colors until April 30, 2017.

** not available as Design Edition

*** not available as Design Edition, not in fashion colors

VenoTrain® discrétion

QUALITIES

The exceptionally fine material of these compression stockings gives them a stylish transparency and lightness.

Available in various designs and versatile colors, these stockings will give you an extra sense of vitality and well-being in your work, leisure and everyday life.

- **highly elastic, making the stockings easy to put on**
- **shapes the legs without constricting them**
- **breathable fine knit for a long-lasting wearing comfort**

MATERIAL CONTENT

Classe 1	10–15 mmHg	75 % polyamide 25 % elastane
Classe 2	15–20 mmHg	70 % polyamide 30 % elastane

CARE INSTRUCTIONS



COLORS

CLAIR

BEIGE

ESPRESSO

BLACK

JAVA









STYLES





VenoTrain discrétion OVERVIEW

SIZING SYSTEM	VenoTrain discrétion Size 1–4 (short and long)
COMPRESSION CLASS	Classe 1 (10–15 mmHg) Classe 2 (15–20 mmHg)
TOE	open, closed

STANDARD SOLUTIONS

	 AD Knee stockings	 AG Thigh-high stockings with silicone lace band	 AG Thigh-high stockings with silicone dotted band	 AT Pantyhose
EDGING TOP BAND				
Clair	● ¹	● ¹	● ¹	
Beige	●	●	●	●
Espresso	●	●	●	●
Black	●	●	●	●
Java	●	●	●	●

CUSTOM-MADE SOLUTIONS

	 AD Knee stockings	 AG Thigh-high stockings with silicone lace band	 AG Thigh-high stockings with silicone dotted band	 AT Pantyhose
EDGING TOP BAND				
Beige	●	●	●	●
Espresso	●	●	●	●
Black	●	●	●	●

¹ only available in classe 2 (15–20 mmHg)



VenoTrain® business

QUALITIES

While traveling or at work: Timelessly elegant with a discreet ribbed appearance, the VenoTrain business meets every requirement for a high-quality knee-high stocking.

The combination of cotton and Timbrelle® high-tech fiber offers top-class wearing comfort with temperature regulation. Highly elastic LYCRA® and the extra-wide supportive band ensure that the stocking is easy to put on and stays perfectly in place.

The new comfort heel, a reinforced sole, and high-tech fibers provide long-lasting freshness and healthy foot conditions.

- **innovative mix of materials**
- **high-quality men's stocking with a classically discreet ribbed structure**
- **reinforced sole**
- **Timbrelle® high-tech fiber for long-lasting freshness around the foot**

COLORS

WHITE

TERRA

BLACK

GRAPHITE

NAVY

MATERIAL CONTENT

- Ccl 1** 80 % polyamide
14 % elastane
6 % cotton
- Ccl 2** 71 % polyamide
24 % elastane
5 % cotton

CARE INSTRUCTIONS



TIMBRELLE®
CARE





STYLES

VenoTrain business OVERVIEW

SIZING SYSTEM	PERFECT-FIT (S-XL)
COMPRESSION CLASS	Ccl 1 (18-21 mmHg) Ccl 2 (23-32 mmHg)
TOE	closed

STANDARD SOLUTIONS

EDGING/TOP BAND	 AD
Top band	●
Shoe size: Men 41-49, Women 36-41	

CUSTOM-MADE SOLUTIONS

EDGING/TOP BAND	 AD
Top band	●

VenoTrain® soft

QUALITIES

Versatile and reliable – for him and for her.

The robust stocking can withstand plenty of wear, yet it is very soft and gentle on the skin.

It is available in many designs with a proven medical effect and a beneficial massage action.

- **for him and her**
- **with a beneficial massage action**
- **particularly gentle on the skin**
- **also available as an individual stocking in natural**

COLORS

NATURAL

CARAMEL

ESPRESSO

BLACK

NAVY

MATERIAL CONTENT

Ccl 1 70 % polyamide
30 % elastane

Ccl 2 65 % polyamide
35 % elastane

CARE INSTRUCTIONS









STYLES


VenoTrain soft OVERVIEW

SIZING SYSTEM	PERFECT-FIT (S–XL)
COMPRESSION CLASS	Ccl 1 (18–21 mmHg) Ccl 2 (23–32 mmHg)
TOE	open, closed

STANDARD SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AGHB	 AT
Top band	●			●
Honeycomb		●		
Silicone dots	●	●		
Garter belt			●	

CUSTOM-MADE SOLUTIONS

EDGING TOP BAND	 AD	 AF	 AG	 AGHB	 AT	 ATU	 ATH	 ATX	 ATE
Top band	●	●			●	●	●	●	●
Lace		●	●						
Honeycomb		●	●						
Silicone dots	●	●	●						
Garter belt				●					

VenoTrain® soft S

QUALITIES

It caters for the special requirements of people who spend prolonged periods of time sitting down, and features a generous heel area and comfortable instep region.

The stocking stimulates circulation using gentle massage action with every movement. The robust knitted fabric can withstand plenty of wear, and can also be easily put on by care staff.

- **wide heel area**
- **adapted to the typical anatomical foot posture while sitting (90 degree angle)**
- **particularly gentle on the skin**

COLORS

NATURAL

CARAMEL

ESPRESSO

BLACK

NAVY

MATERIAL CONTENT

Ccl 1 70 % polyamide
30 % elastane

Ccl 2 65 % polyamide
35 % elastane

CARE INSTRUCTIONS







STYLES



VenoTrain soft S OVERVIEW

SIZING SYSTEM	PERFECT-FIT (XS–XL)
COMPRESSION CLASS	Ccl 1 (18–21 mmHg) Ccl 2 (23–32 mmHg)
TOE	open, closed

STANDARD SOLUTIONS*

EDGING TOP BAND	 AD	 AG
Top band	●	
Silicone dots	●	●

CUSTOM-MADE SOLUTIONS

EDGING TOP BAND	 AD	 AG
Top band	●	
Honeycomb		●
Silicone dots	●	●

* available in color caramel

VenoTrain® clinic

QUALITIES

Developed for the specific requirements of modern treatment with compression stockings.

With a simplified sizing system for quick and efficient treatment. Discreet appearance for him and her, gentle on the skin and comfortable to wear. For medical effectiveness that makes no compromises.

- **soft, durable material**
- **flat-pack foil bag packaging**
- **four standard sizes, each in two lengths**
- **also available as a single stocking**

MATERIAL CONTENT

- Ccl 1** 77 % polyamide
23 % elastane
- Ccl 2** 74 % polyamide
26 % elastane

CARE INSTRUCTIONS



COLORS

NATURAL






STYLES




VenoTrain clinic OVERVIEW

SIZING SYSTEM	VenoTrain clinic (S-XL)
COMPRESSION CLASS	Ccl 1 (18-21 mmHg) Ccl 2 (23-32 mmHg)
TOE	open, closed

STANDARD SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AT*
Top band	●		●
Silicone dots		●	

CUSTOM-MADE SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AT*
Top band	●		●
Silicone dots		●	

* available from 01.03.2017

VenoTrain® impuls

QUALITIES

The high working pressure, even during small muscle movements, produces a really deep action. Easy to put on and breathable thanks to microfibers.

Highly effective for advanced vein problems, post-thrombotic syndrome, providing follow-up care for healed ulcers and providing therapy in the early stages of lymphedema.

- **short-stretch material with skin-friendly microfiber**
- **highly differentiated V6+ sizing system**

COLORS

CARAMEL

BLACK

MATERIAL CONTENT

Ccl 2 80 % polyamide
20 % elastane

Ccl 3 75 % polyamide
25 % elastane

CARE INSTRUCTIONS








STYLES







VenoTrain impuls OVERVIEW

SIZING SYSTEM	V6+ (I – IV)
COMPRESSION CLASS	Ccl 2 (23 – 32 mmHg) Ccl 3 (34 – 46 mmHg)
TOE	open, closed

STANDARD SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AGHB
Top band	●		
Silicone dots	●	●	
Garter belt			●

CUSTOM-MADE SOLUTIONS

EDGING TOP BAND	 AD	 AG	 AGHB	 AT	 ATH	 ATX
Top band	●			●	●	●
Silicone dots	●	●				
Garter belt			●			

VenoTrain® ulcertec

QUALITIES

The combination of an overstocking with a patented rhomboid knit and a skin-friendly liner for fixing the wound dressing in place makes the item easy to put on without any help from others.

The liners are also designed to provide a continuous low level of compression during the night, which can be individually ordered as a set of three and are suitable for washing at high temperatures.

- **easy to put on**
- **constant pressure**
- **discreet appearance**
- **X5+ sizing system**

COLORS

NATURAL

(Overstocking)

WHITE

(Liner)

MATERIAL CONTENT

39 – moderate 87 % polyamide
13 % elastane

46 – strong 69 % polyamide
31 % elastane

CARE INSTRUCTIONS FOR OVERSTOCKING



CARE INSTRUCTIONS FOR LINER





STYLES

VenoTrain ulcertec OVERVIEW

SIZING SYSTEM	X5+ (XS–XL)
COMPRESSION CLASS	39 – moderate (34–46 mmHg) Liner with gentle compression 46 – strong (34–46 mmHg) Liner with moderate compression
TOE	Overstocking – open Liner – closed

STANDARD SOLUTIONS

EDGING/TOP BAND	 AD
Top band	●

CUSTOM-MADE SOLUTIONS

EDGING/TOP BAND	 AD
Top band	●

VenoTrain ulcertec, 39 – moderate

- compression strength 34–46 mmHg
- Liner with gentle compression
- venous ulcer of the leg with good tendency to heal
- light liner for patients who can only tolerate a little compression at night

VenoTrain ulcertec, 46 – strong

- compression strength 34–46 mmHg
- Liner with moderate compression
- stagnating ulcers with a poor tendency to heal
- Liner with moderate compression providing wearing tolerance and a good acceptance level of compression at night

VenoTrain® soft Arm

QUALITIES

Seamless with an extra soft inner elbow area. This facilitates everyday mobility combined with complete medical efficacy.

- **Comfort crook of the elbow for greater freedom of movement, wearing comfort and a reduced pressure sensation**
- **noticeable effect throughout the entire wearing period**
- **gentle on the skin, soft and breathable**

COLORS

NATURAL

CARAMEL

BLACK

MATERIAL CONTENT

- Ccl 1** 74 % polyamide
26 % elastane
- Ccl 2** 69 % polyamide
31 % elastane

CARE INSTRUCTIONS





STYLES

VenoTrain soft arm sleeves
OVERVIEW

SIZING SYSTEM	VenoTrain soft Arm (S-XL)
COMPRESSION CLASS	Ccl 1 (18–21 mmHg) Ccl 2 (23–32 mmHg)

STANDARD SOLUTIONS

EDGING/TOP BAND	CG
Top band	●
Silicone dots	●

VenoTrain® delight

QUALITIES

Available as a flat-knit leg stocking with skin-friendly, latex-free materials and exceptional fit. The special gripRelief® structure at the top ensures the item holds comfortably and securely in place.

For the treatment of advanced venous and lymphatic symptoms.

- **breathable double-stretch knit**
- **extra-flat seams**
- **wide variety of styles**

MATERIAL CONTENT

Ccl 2	80 % polyamide 20 % elastane
Ccl 3	79 % polyamide 21 % elastane
Ccl 4	75 % polyamide 25 % elastane

CARE INSTRUCTIONS



COLORS

CARAMEL

BLACK






STYLES












VenoTrain delight OVERVIEW

SIZING SYSTEM	VenoTrain delight
COMPRESSION CLASS	Ccl 2 (23–32 mmHg) Ccl 3 (34–46 mmHg) Ccl 4* (from 49 mmHg)
TOE	open diagonal cut closed

STANDARD SOLUTIONS

EDGING TOP BAND	 AD
Top band	●
Silicone dots	●

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CUSTOM-MADE SOLUTIONS

EDGING TOP BAND											
Top band	●				●	●	●	●	●	●	●
Silicone dots	●	●	●				●	●		●	
Garter belt				●							
Additional features											
T-heel	●	●	●	●	●						
Visco-elastic pads	●	●	●	●							

* only custom-made solutions (AD, AF and AG)

VenoTrain® curaflow

QUALITIES

The high microfiber content and the innovative way these are processed make this item unique. Extra-flat sensitive stitching and special comfort zones ensure this flat knit treatment provides excellent wearing comfort.

Available for the leg, foot, arm, hand and thorax.

- **smooth microfibers make the item easy to put on**
- **breathable knitted fabric, extra-soft toe**
- **anatomical, extra-soft area behind the knee for unrestricted freedom of movement**
- **special shapes to suit individual requirements**

COLORS

CREAM

CARAMEL

BLACK

NAVY

Also from
March 1, 2017

BORDEAUX

ANTHRACITE

MATERIAL CONTENT

- Ccl 1 – 2** 73 % polyamide
27 % elastane
- Ccl 3** 70 % polyamide
30 % elastane
- Ccl 4** 65 % polyamide
35 % elastane

CARE INSTRUCTIONS






STYLES

VenoTrain curaflow OVERVIEW


SIZING SYSTEM	Custom-made
COMPRESSION CLASS	LEG Ccl 1–4 ARM Ccl 1–3 THORAX Ccl 1
TOE	open Diagonal cut closed closes at an angle

CUSTOM-MADE SOLUTIONS


LEG AND TOE

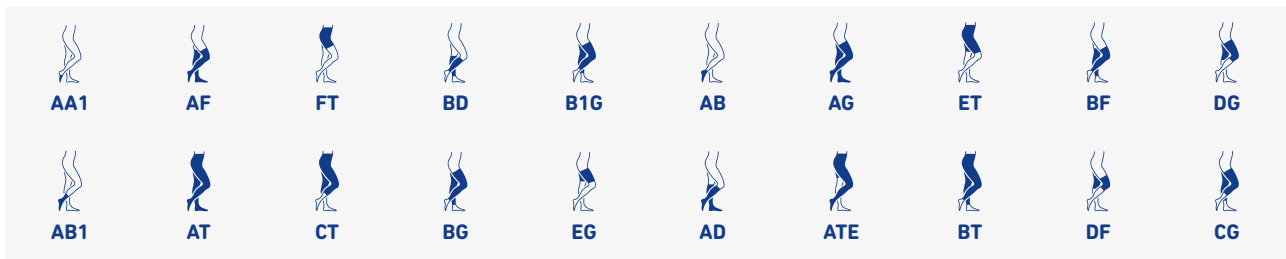

ARM AND HAND

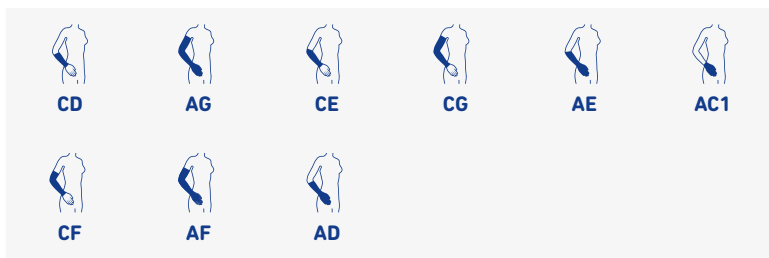

THORAX

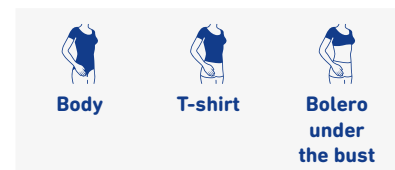

STANDARD PRODUCT RANGE

ROUND-KNIT PRODUCTS

Specifications		Size	Compression class (Ccl)			Edging/Top band						Toe	
			1	2	3	Top band	Spitze sensitive**	Honeycomb	Silicone dots	Silicone dot pattern	Garter belt	open	closed
VenoTrain			•	•									
look	AD	S–XL	•	•		•							•
	AG	S–XL	•	•					•				•
	AT	S–XL	•	•		•							•
	ATU	S–XL	•	•		•							•
micro	AD	S–XL	•	•		•			•*			•	•
	AG	S–XL	•	•			•		•*	•		•	•
	AT	S–XL	•	•		•						•	•
	ATU	S–XL	•	•		•						•	•
micro Design Edition: Jive, Tango	AD	S–XL	•	•		•							•
	AG	S–XL	•	•			•		•				•
	AT	S–XL	•	•		•							•
business	AD	S–XL	•	•		•							•
soft	AD	S–XL	•	•		•			•			•	•
	AG	S–XL	•	•			•		•			•	•
	AGHB	S–XL	•	•						•		•	•
	AT	S–XL	•	•		•						•	•
soft S	AD	XS–XL	•	•		•			•			•	•
	AG	XS–XL	•	•					•			•	•
clinic	AD	S–XL	•	•		•						•	•
	AG	S–XL	•	•					•			•	•
	AT	S–XL	•	•		•						•	•
impuls	AD	I–VI		•	•	•			•			•	•
	AG	I–VI		•	•				•			•	•
	AGHB	I–VI		•	•					•		•	•
ulcertec, 39 moderate	AD	XS–XL	34–46 mmHg			•						Over-stocking	Liner
	AD	XS–XL	34–46 mmHg			•						Over-stocking	Liner

* not available in fashion colors

** From May 1, 2017, the following will apply to VenoTrain micro:

The Lace Sensitive silicone top band will replace the Lace silicone top band.

The Lace Sensitive silicone top band will be available only for the Design Edition and fashion colors until April 30, 2017.

Please refer to the current price list for information on further styles.

CUSTOM PRODUCT RANGE

ROUND-KNIT PRODUCTS


Specifications		Compression cl. (Ccl)			Edging/Top band							Toe				Pantyhose top
		1	2	3	Top band	Lace sensitive**	Lace	Honeycomb	Silicone dots	Silicone dot pattern	Garter belt	open	closed	No foot	Stump	Open crotch
VenoTrain																
look	AD	●	●		●								●			
	AG	●	●				●					●				
	AT	●	●		●							●				
	ATU	●	●		●							●		●		
micro	AD	●	●		●				●			●	●			
	AF	●	●		●	●			●	●		●	●			
	AG	●	●		●	●			●	●		●	●			
	AT	●	●		●							●	●			
	ATU	●	●		●							●	●			
micro Design Edition: Jive, Tango	AD	●	●		●							●	●			
	AG	●	●			●				●		●	●			
	AT	●	●		●							●	●			
business	AD	●	●		●				●			●	●			
soft	AD	●	●		●				●			●	●	●	●	
	AF	●	●		●		●	●	●			●	●	●	●	
	AG	●	●				●	●	●			●	●	●	●	
	AGHB	●	●								●	●	●	●	●	
	AT	●	●		●							●	●	●	●	●
	ATU	●	●		●							●	●	●		
	ATH	●	●		●							●	●	●		
	ATX	●	●		●							●	●	●		
	ATE	●	●		●							●	●	●		
soft S	AD	●	●		●				●			●	●			
	AG	●	●				●	●				●	●			
clinic	AD	●	●		●							●	●			
	AG	●	●						●			●	●			
	AT	●	●		●							●	●			
impuls	AD		●	●	●				●			●	●			
	AG		●	●					●			●	●			
	AGHB		●	●							●	●	●			
	AT		●	●	●							●	●			
	ATH		●	●	●							●	●			
	ATX		●	●	●							●	●			
ulcertec, 39 moderate	AD	34–46 mmHg			●						Over-stocking	Liner				
ulcertec, 46 strong	AD	34–46 mmHg			●						Over-stocking	Liner				

Please refer to the current price list for information on further styles.

Custom product range Lymphology products – please refer to VenoTrain soft Arm (page 24–25) | VenoTrain delight (page 26–27) | VenoTrain curaflow (page 28–29)

Sizing system Perfect-Fit round-knit products

VenoTrain look | VenoTrain micro | VenoTrain business | VenoTrain soft | VenoTrain soft S

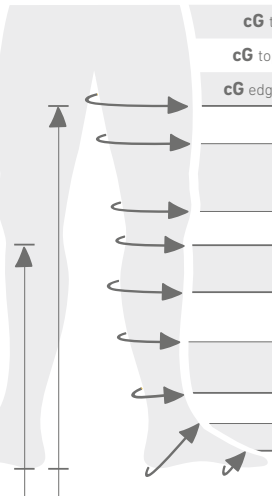


Circumference	XS*	S		M		L		XL	
	Normal	Normal	Plus	Normal	Plus	Normal	Plus	Normal	Plus
cG top band, wide		51-57	59-66	56-63	62-69	60-67	66-74	63-71	71-80
cG top band, standard	44-50	45-51	52-59	49-56	55-62	53-60	58-66	55-63	62-71
cG edging, AT, top band**		45-57	52-66	49-63	55-69	53-67	58-74	55-71	62-80
cF	39-50	40-52	43-55	43-55	46-60	45-59	50-64	49-63	52-68
cE	32-38	33-40	36-44	35-43	39-49	38-48	42-53	42-53	47-58
cD	28-31	29-35	32-38	32-38	35-42	34-42	37-45	37-45	41-49
cC	29-33	31-37	34-40	34-40	37-44	37-45	39-47	40-48	43-51
cB1	23-26	25-29	27-31	27-32	29-34	29-35	31-37	32-38	34-40
cB	18-20	20-22		22-25		25-28		28-31	
cY	27-31	29-33		31-35		34-40		37-44	
cA	17-22	19-24		22-27		25-31		28-35	

* only for VenoTrain soft S – limited product range
** Top band lace sensitive

Length	short	long	Type
ℓG	61-70	70-80	AG, AT
ℓD	35-40	40-46	AD

Sizing system X5+ VenoTrain ulcer tec

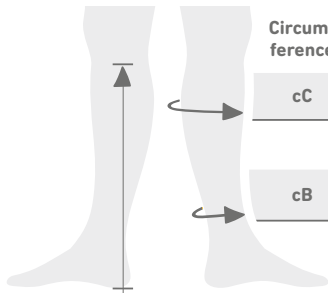


Circumference	XS		S		M		L		XL	
	Normal	Plus	Normal	Plus	Normal	Plus	Normal	Plus	Normal	Plus
cC	24-29	28-34	27-32	31-38	29-36	35-42	33-40	38-47	36-44	43-52
cB	20-21,5		21,5-23		23-25		25-28		28-32	

Length	short	long	Type
ℓD	35-40	40-45	AD

Sizing system

VenoTrain clinic

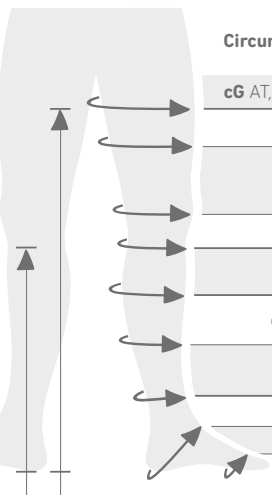


Circumference	S		M		L		XL	
	Normal	Plus	Normal	Plus	Normal	Plus	Normal	Plus
cG AT, top band	46-55	55-64	49-58	58-67	52-62	62-72	55-66	66-78
cF	40-52	43-55	43-55	46-60	45-59	50-64	49-63	52-68
cE	33-40	36-44	35-43	39-49	38-48	42-53	42-53	47-58
cD	29-38		32-42		34-45		37-49	
cC	31-40		34-44		37-47		40-51	
cB1	25-31		27-34		29-37		32-40	
cB	20-22		22-25		25-28		28-31	
cY	29-33		31-35		34-40		37-44	
cA	19-24		22-27		25-31		28-35	

Length	short	long	Type
ℓG	61-70	70-80	AG, AT
ℓD	35-40	40-46	AD


Sizing system V6+

VenoTrain impuls

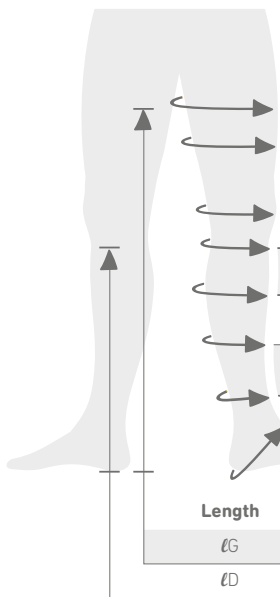

Circumference	I		II		III		IV		V		VI	
	Normal	Plus	Normal	Plus	Normal	Plus	Normal	Plus	Normal	Plus	Normal	Plus
cG	45-53	51-59	48-56	54-62	51-59	57-66	54-63	60-69	57-66	63-73	60-70	66-76
cF	39-45	43-50	42-49	46-53	45-52	49-57	48-56	52-60	51-59	55-64	54-63	58-67
cE	32-37	35-40	34-39	37-42	36-41	39-44	38-44	41-47	40-46	43-49	42-48	45-51
cD	28-32	30-34	30-34	32-37	32-37	34-39	34-39	36-41	36-41	38-43	38-43	40-46
cC	29-33	31-35	31-35	33-38	33-38	35-40	35-40	37-42	37-42	39-44	39-44	41-47
cB1	23-27	24-28	25-29	26-30	27-31	28-32	29-34	30-35	31-36	32-37	33-38	34-39
cB	19-20,5		20,5-22		22-24		24-26		26-28,5		28,5-31	
cY	25-29		26-31		28-33		30-35		32-38		35-41	
cA	19-22		20-23		22-25		24-27		26-30		28-33	
Length	short		long		Type							
ℓG	62-70		70-80		AG							
ℓD	35-40		40-44		AD							

Sizing system VenoTrain delight

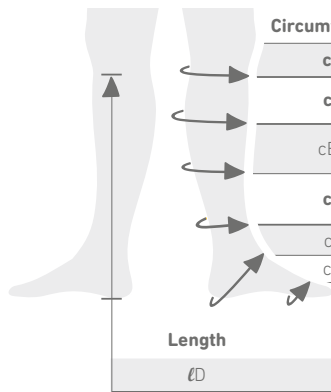


Circumference	I	II	III	IV	V	VI
cD	28-34	30-37	32-39	34-41	36-43	38-46
cC	29-35	31-38	33-40	35-42	37-44	39-47
cB1	23-28	25-30	27-32	29-35	31-37	33-39
cB	19-20,5	20,5-22	22-24	24-26	26-28,5	28,5-31
cY	25-29	26-31	28-33	30-35	32-38	35-41
cA	19-22	20-23	22-25	24-27	26-30	28-33
Length	short	long	Style			
ℓD	35-39	40-44	Knee-high (AD)			

Product range: Knee-high (AD), 20-30 mmHg (Ccl2 + Ccl3).
Open-toe, Caramel, with or without silicone dotted border.
Only available in selected countries.

Sizing system VenoTrain discrétion

Size	1	2	3	4
cG	42-56	49-63	56-70	63-77
cC	28-36	32-40	35-43	38-46
cB	18-21	21-24	24-27	27-31
	short		long	
AG AT	< 70		> 70	
AD	< 38		> 38	

Sizing system VenoTrain soft Arm

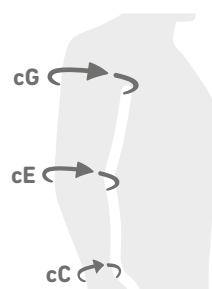

	S	M	L	XL
	min - max	min - max	min - max	min - max
cG	24-30	27-33	29-36	32-40
cE	22-26	24-29	27-32	30-35
cC	14-15	15-16,5	16,5-18	18-20
Length	short	long	Type	
ℓD	39-44	44,1-50	CG	

VenoTrain® act

QUALITIES

These high quality support stockings provide comfortable stimulation. They offer gentle support for feelings of heaviness and swelling in your legs, preventing circulatory disorders. They ensure that legs stay fit for longer when sitting or standing for prolonged periods, and when traveling. VenoTrain act is available in the versions "sheer elegance", "cotton" and "microfiber", combined with modern materials, and in attractive colors – for him and for her.

CARE INSTRUCTIONS



COLORS

SAND

CARMEL

AMARETTO

ESPRESSO

BLACK

GRAPHITE

NAVY

SHEER ELEGANCE








- elegant and transparent
- in different colors
- available as a knee-high or thigh-high stocking with a silicone lace band, and as pantyhose
- also available as pantyhose for pregnant women with a special panty area that expands as the mother's bump grows

MATERIAL CONTENT


81 % polyamide
19 % elastane

STYLES


					
	AD	AG	AT	AT CAL	ATU
SIZE	1–4	1–5	1–5	2–5	1–5
DEN	70/140	70/140	70/140	70/140	140
	Sand	●	●	●	
	Caramel	●	●	●	●
	Amaretto	●	●	●	
COLORS	Espresso	●	●	●	
	Black	●	●	●	●
	Graphite	●	●	●	
	Navy	●	●	●	


SIZING SYSTEMS (SHEER ELEGANCE)

AG, AT, ATU

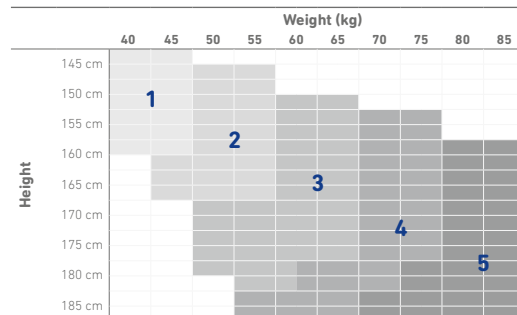

AT CAL

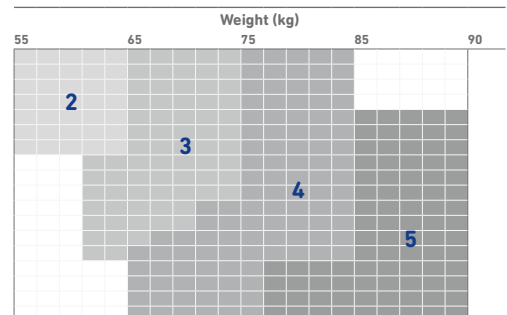

SIZE	AD	1	2	3	4
SHOE SIZE		37–39	39–41	41–43	43–45



COTTON



- particularly high level of wearing comfort based on cotton
- with distinctive ribbed look
- opaque
- available as a highly supportive knee-high stocking (280 denier)
- in four colors
- unisex

MATERIAL CONTENT

27 % polyamide
10 % elastane
63 % cotton

MICROFIBER




- particularly soft and gentle on the skin, thanks to the high microfiber content
- breathable
- available as a highly supportive knee-high stocking (280 denier)
- in four colors
- unisex


MATERIAL CONTENT

94 % polyamide
6 % elastane



STYLES

	 AD
SIZE	1-4
DEN	280
COLORS	Nature <input type="checkbox"/>
	Black <input type="checkbox"/>
	Graphite <input type="checkbox"/>
	Navy <input type="checkbox"/>

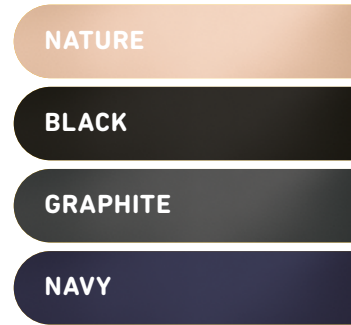COLORS



SIZING SYSTEM (COTTON, MICROFIBER)

SIZE	AD	1	2	3	4
SHOE SIZE	 WOMAN	37-39	39-41	41-43	43-45
	 MEN	39-41	41-43	43-45	45-47

VenoTrain® Accessories

THE PERFECT ADDITION FOR ALL VENOTRAIN PRODUCTS

The handy extras from the VenoTrain accessories range are useful for ensuring that patients feel comfortable using and wearing their VenoTrain compression stockings. They provide effective support for treatment and are easy to handle and care for the compression stockings as well as your customers' skin.

VenoTrain® glider

A PRACTICAL AID

VenoTrain glider is the practical aid for putting on and taking off your medical compression stockings.

The lightweight, hard-wearing material makes the stocking glide effortlessly over your arm or leg. It is suitable for compression stockings with open or closed toe.

Saving space and time, it is the ideal companion when you're out and about.

- extremely lightweight, robust, and tear-resistant
- easy to handle
- also ideal for travel



VenoTrain® glider plus


A ROBUST AID

VenoTrain glider plus is a robust donning aid for all types of medical compression stockings.

It has been tested by an independent institute for ease of use, even for customers with restricted mobility.

VenoTrain glider plus features a particularly compact design.

- compact, ergonomic design
- light metal frame with folding handles
- easy to handle without the need for any great effort

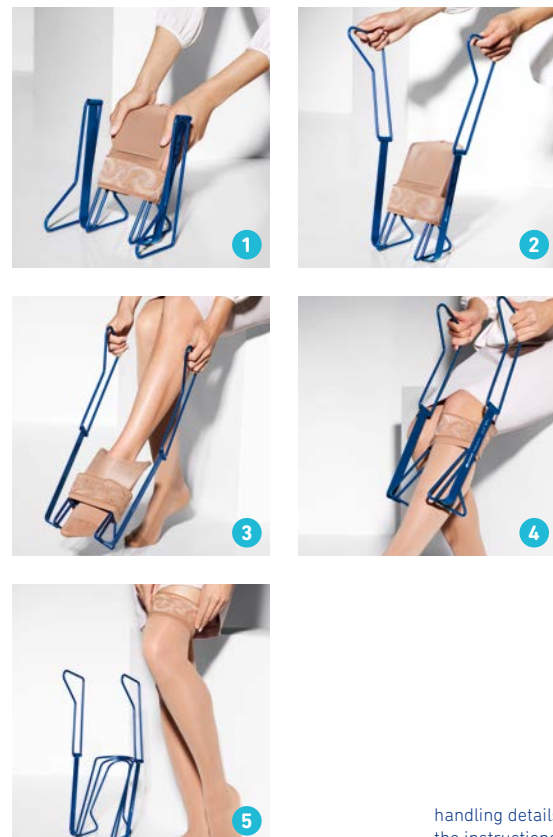

handling details provided in the instructions for use



Rubber gloves

help with the correct donning of the compression stockings



Special detergent*

ensures careful cleaning and provides lasting protection for your products



Lingerie bag

to protect your compression stockings during machine wash



VenoTrain® lotion*

developed in collaboration with Sebapharma; contains chamomile extract and is specially designed for people with sensitive skin



Care foam*

contains panthenol to protect and soothe particularly stressed and sensitive skin



Starter kit

including gloves, a foot file, lingerie bag, non-skid soles, an exercise band, and loyalty card

* only available in selected countries

INDIVIDUAL CUSTOMER SERVICE

ADDITIONAL SERVICES

STANDARD SOLUTIONS

	look	micro	micro Design Edition	business	soft	softS	clinic	impuls
Batik navy		●						
Batik black		●						

CUSTOM-MADE SOLUTIONS

	look	micro	micro Design Edition	business	soft	softS	clinic	impuls	delight
Lengthen or shorten the knitted toe area		●			●	●		●	
Toe – seam on the outside	●	●	●	●	●	●	●	●	●
Toe – make wider, narrower, anatomic		●	●	●	●	●	●	●	●
Panty area at the front/back – 2 seams	●	●			●	●		●	
AT with men's gusset (longitudinal slit)					●	●		●	
AT with an extra-wide gusset for the fuller figure	●	●			●	●		●	
Sew in a foam pad									●
Integration of stoma opening					●				
Batik navy		●							
Batik black		●							

ORDER PADS

We have developed a range of customer order pads for ordering standard and custom-made solutions. With these pads, the required version of a particular VenoTrain product can be ordered quickly and easily. We are happy to send you these order pads free of charge.

The different order sheets can also be downloaded at any time:

www.bauerfeind.com/downloads

STANDARD ORDER PAD		Item number
VenoTrain		27120200000005
CUSTOM-MADE ORDER PADS		Item number
Circular knit products	VenoTrain	27120200001003
Flat knit products	VenoTrain delight leg	27120200000003
	VenoTrain delight arm	27120200001005
	VenoTrain delight glove	27120200001006
	VenoTrain delight toe caps	27120200001007
	VenoTrain curaflow leg	27142207001001
	VenoTrain curaflow toes	27142207001002
	VenoTrain curaflow arm/hand	27142207001003
	VenoTrain curaflow thorax	27141203001003

PRODUCTS AND HEALTH CARE CONCEPTS: FOR BETTER QUALITY OF LIFE

As one of the world's leading manufacturers of medical products, Bauerfeind AG has, through its products and service concepts, been making a significant contribution toward maintaining and restoring health and increasing well-being for more than 85 years – for a better quality of life.

Our specialists work with physicians and scientists from all over the world and are fully aware of what is expected of them, in terms of both the tradition and reputation of a family-run business and the high performance levels demanded by today's healthcare market.

Numerous patents and international awards underline our company's flair for innovation and our outstanding expertise in technology and manufacturing. We intend to continue delivering quality "Made in Germany".

Supports and orthoses

Whether for prevention, treating symptoms or providing post-operative care, physicians and physiotherapists around the world trust our supports and orthoses, which are available in a wide range of styles for all relevant forms of treatment and set the standard for maximizing therapeutic success.

Insoles

In the orthopedic orthosis segment, we set the standard with traditional products and innovative foot orthoses concepts such as TRlactive and ErgoPad. Comfort and therapeutic shoes and pressure relief orthoses round off a comprehensive range for modern medical foot care.

Measurement technology

Medical products will only work if they fit perfectly. With this in mind, Bauerfeind has specifically developed digital measurement systems for medical retailers to improve treatment with medical compression stockings and to enable the manufacture of custom-made foot orthoses.

If you have any questions, ideas or requests, please contact us:

BAUERFEIND AG

Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-10 00
F +49 (0) 36628 66-19 99
E info@bauerfeind.com

AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

Bauerfeind Benelux B.V.
Waarderveldweg 1
2031 BK Haarlem

THE NETHERLANDS

P +31 (0) 23 531-9427
F +31 (0) 23 532-1970
E info@bauerfeind.nl

BELGIUM

P +32 (0) 2 527-4060
F +32 (0) 2 792-5345
E info@bauerfeind.be

BOSNIA AND HERZEGOVINA

Bauerfeind d.o.o.
Meše Selmovića 19
71000 Sarajevo
P +387 (0) 33 710-100
F +387 (0) 33 619-422
E info@bauerfeind.ba

CROATIA

Bauerfeind d.o.o.
Goleška 20
10020 Zagreb
P +385 (0) 1 6542-855
F +385 (0) 1 6542-860
E info@bauerfeind.hr

FRANCE

Bauerfeind France S.A.R.L.
B.P. 50258
95957 Roissy CDG Cedex
P +33 (0) 1 4863-2896
F +33 (0) 1 4863-2963
E info@bauerfeind.fr

ITALY

Bauerfeind AG
Filiale Italiana
Piazza Don Enrico Mapelli 75
20099 Sesto San Giovanni (MI)
P +39 02 8977 6310
F +39 02 8977 5900
E info@bauerfeind.it

NORDIC

Bauerfeind Nordic AB
Kungsholmstorg 8
11221 Stockholm
P +46 (0) 774 100-020
F +46 (0) 774 100-021
E info@bauerfeind.se

REPUBLIC OF MACEDONIA

Bauerfeind Dooel Skopje
50 Divizija 24 a
1000 Skopje
P +389 (0) 2 3179-002
F +389 (0) 2 3179-004
E info@bauerfeind.mk

SERBIA

Bauerfeind d.o.o.
102 Omladinskih brigada
11070 Novi Beograd
P +381 (0) 11 2287-050
F +381 (0) 11 2287-052
E info@bauerfeind.rs

SINGAPORE

Bauerfeind Singapore Pte Ltd.
Blk 41 Cambridge Road #01-21
Singapore 210041
P +65 6396-3497
F +65 6295-5062
E info@bauerfeind.com.sg

SLOVENIA

Bauerfeind d.o.o.
Dolenjska cesta 242 b
1000 Ljubljana
P +386 (0) 1 4272-941
F +386 (0) 1 4272-951
E info@bauerfeind.si

SPAIN

Bauerfeind Ibérica, S.A.
Historiador Diago, 13
46007 Valencia
P +34 96 385-6633
F +34 96 385-6699
E info@bauerfeind.es

SWITZERLAND

Bauerfeind AG
Vorderi Böde 5
5452 Oberrohrdorf
P +41 (0) 56 485-8242
F +41 (0) 56 485-8259
E info@bauerfeind.ch

UNITED ARAB EMIRATES

Bauerfeind Middle East FZ-LLC
GF-09, District 1,
71 MPC Building
Dubai Healthcare City
P.O. Box 505116
Dubai
P +971 4 4335-685
F +971 4 4370-344
E info@bauerfeind.ae

UNITED KINGDOM

Bauerfeind UK
Regents Place
338 Euston Road
London
NW1 3BT
P +44 (0) 121 446-5353
F +44 (0) 121 446-5454
E info@bauerfeind.co.uk

USA

Bauerfeind USA, Inc.
1230 Peachtree St NE
Suite 1000
Atlanta, GA 30309
P +1 800 423-3405
P +1 404 201-7800
F +1 404 201-7839
E info@bauerfeindusa.com